

## 20 Disposable Ophthalmic Gauzes

Specifically for the prevention of infections of the periocular area in adults and children

### What OZOLID<sup>®</sup> is and what it is for

OZOLID<sup>®</sup> ophthalmic gauzes is a Medical Device for effective mechanical removal of secretions, whether lipidic or dried, cellular residues and other contaminants from eyelids and eyelashes which are released in inflammatory conditions such as blepharitis, blepharoconjunctivitis, styes, chalazia or in cases of periocular trauma.

Additionally, thanks to the ozonated oil and liposomes, OZOLID<sup>®</sup> promotes the breakdown of the biofilm, a natural defence mechanism for bacteria which can form along the eyelid margins. The accumulation of biofilm can cause and worsen inflammatory and infectious processes affecting the eyelids. Therefore, removing the biofilm helps to reduce inflammatory symptoms caused by the action of the bacteria themselves. Removing secretions and bacterial biofilm from eyelids and eyelashes is also helpful before and after ocular surgery, as it prevents and reduces the risk of postoperative infections caused by the proliferation of microorganisms.

OZOLID<sup>®</sup> is indicated:

- for blepharitis and blepharoconjunctivitis, infectious or allergic, styes and chalazia, to remove secretions from the eyelids and eyelashes, thus providing relief from symptoms;
- before and after ophthalmological surgery to reduce infections caused by bacteria present on the eyelids and eyelashes.

Because of the high tolerability of its components, OZOLID<sup>®</sup> can be used daily to protect the periocular area.

### Composition

NWF gauze soaked in a solution containing: LipozonEye<sup>®</sup> (Self-preserving ozonated vegetable oil, Liposomes), Hydroxypropyl methylcellulose, Boric acid, Sodium tetraborate, PHMB, Purified water.

### Directions of use

After thoroughly washing your hands, open the sachet and take out the gauze. Gently wipe the gauze over the eyelid area, making a few light, circular motions to allow adequate removal of ocular secretions and any other deposits (pollution, dust, etc.). For hard-to-remove secretions or inflammation affecting the palpebral edge (blepharitis, blepharoconjunctivitis, styes, chalazia), we recommend warming the closed sachet for a few minutes in water (at 40°C), then using the gauze as a warm compress. For swelling of the eyelid, we recommend chilling the closed sachet in the refrigerator for at least one hour, then applying the gauze to the closed eyelid for a few minutes.

### Warnings

Use one gauze for each eye. Keep out of the reach of children. Discontinue treatment if there is obvious irritation or hypersensitivity to the product. The functionality and safety of the product are only ensured if the packaging is undamaged. Follow your doctor's instructions carefully. Never reuse gauze, throw it away immediately after use. For external use. Dispose of properly after use. Read the package leaflet carefully before use.

### Storage

Store in a cool, dry place away from heat sources.

### Packaging

Contains 20 disposable NWF gauzes.

EXTERNAL USE