

LACRICOMPLEX®

MULTIMOLECULAR LUBRICANT AND PROTECTIVE OPHTHALMIC SOLUTION

with GlicoPro®

10 ml

The dry eye is a rather frequent pathology of the ocular surface, due to various causes and characterised by

- alteration and instability of the tear film
- inflammation and ocular surface lesions
- corneal nerve fibre abnormalities

which are accompanied by various symptoms, such as burning, foreign body sensation, blurred vision, photophobia.

LACRICOMPLEX® is a lubricating, protective, moisturising, soothing and antioxidant ophthalmic solution, containing **GlicoPro®**, a multimolecular complex based on proteins, sulfur and non-sulfur glycosaminoglycans contained in a mucinic base consisting of hydroxypropyl methylcellulose (HPMC). Glycosaminoglycans, in particular, constitute an important part of the glycocalyx of the cell surface and can represent ligands for many molecules like growth factors, cytokines, chemokines, which regulate the inflammatory response.

LACRICOMPLEX®, therefore, contains a triple mucin component together with a mucoprotein component and a protein, with a lubricating, moisturising, antioxidant and protective action that is useful in the treatment of dry eyes, also as a coadjuvant in the presence of irritation and inflammation caused by environmental conditions or mechanical stress.

LACRICOMPLEX® also promotes the physiological corneal reparative processes and supports the restoration of the functionality of the corneal nerve endings, which decreases in case of dry eye syndrome.

The ophthalmic solution is contained in a technological bottle which avoids the use of preservatives and allows an easy dispensing of the full volume of the liquid.

INDICATIONS

LACRICOMPLEX® thanks to the lubricating, protective, moisturising, soothing and antioxidant action of its components, allows to treat the symptoms of dry eyes, also as an adjuvant in the presence of irritation and inflammation caused by environmental conditions or mechanical stress.

In addition, **LACRICOMPLEX®** is useful in promoting physiological corneal repair processes and supports the regeneration of corneal nerve endings, altered during eye dryness.

HOW TO USE

Apply one or two drops in each eye 3-4 times a day or as advised by your doctor. The product can be used while wearing contact lenses.

COMPOSITION

GlicoPro 10%, HPMC 0.2%, boric acid 0.25%, sodium tetraborate 0.03%, sodium chloride 0.810%, q.s. purified water at 100 ml.

WARNINGS

- Wash your hands thoroughly before using the product.
- Do not use in case of known allergy and hypersensitivity to the components of the product.
- Do not touch the ocular surface with the dropper nozzle.
- Close the bottle tightly after use.
- Do not use the product after 60 days from opening it.
- Do not use the product after the expiry date.
- Administer the product at least 5 minutes before or after administration of other topical ophthalmic products.
- Store the product at a temperature between 5°C and 25°C.
- In case of adverse reactions, discontinue use of the product and consult a specialist.
- Do not swallow.
- Keep away from heat sources.
- Keep out of reach of children.
- Do not use the product if the packaging appears damaged.



STERILE A



REF 10031700IT
00041700IT
10051710IT
00061710IT

LACPLEXengF000422 - Rev. 08/2021